

WE
SPECIALIZE
IN ENDO.

ENDODONTIC INSTRUMENTS

Internal Information Catalogue

* Product Manual Document Rev.4



Shenzhen Superline Technology Co.,Ltd.

Address: Room 302, Building A4, Nanshan Tel: +86 755-2601 6156

Zhiyuan, No.1001,Xueyuan Avenue

Nanshan District, Shenzhen

Guangdong, China 518000

Fax: +86 755-2601 6026

E-mail: sales@supline.com

Website: www.medental-usa.com

SuperFile®

ENDODONTIC INSTRUMENTS

Internal Information Catalogue

Table of Contents	01
	02
Company Introduction	03
	04
Super-Pro	06
Super-Wave Gold	07
Super-Wave Tunnel	07
RC-Series	08
Taper Next	09
Master	10
RS File	11
Pedo File	12
S Taper	13
Super-Pass	14
Super-Large	15
Super-One	15
Super-Can	16
TN File	16
S7 File	17
CF File	17

S Taper Ultimate File	18
RT Series	18
Path File	19
AS Series	19
S Path	20
Superfile	21
C File	21
K File	22
H File	22
K-Flex	23
Reamer	23
Niti Engine Condenser	23
S.S. Rotary Paste Carrier	24
Spreader	24
Plugger	24
Smooth Broach	25
Reamer	25
Barbed Broach	25
Endo Debrider	26
Endo Opener	26
Gates Drill	27
Peeso Reamer	27
GP Plugger	28
Purchase order code	29+

COMPANY PROFILE

2002 year

Founded in

10,000 m²

Modern factory area



SuperFile® was founded in 2002.

With over two decades of development, **SuperFile®** has become one of the most renowned dental product manufacturers in China, exporting dental instruments under the SuperFile brand to markets worldwide.

Established by three experts in material science and technology, **SuperFile®** focuses on the development and innovation of shape memory alloy nickel-titanium, dedicating efforts to applying outstanding Ni-Ti metal performance to improved dental treatment devices and introducing new endodontic and orthodontic products for dentists worldwide.

SuperFile® products are precisely manufactured at the micrometer level under strict inspection standards in a modern factory equipped with hundreds of advanced machines to ensure products meet top-quality requirements. Our products are certified with **CE 0197** and **FDA**, and the quality system has passed **ISO 13485**.

SuperFile® strives to invest and innovate in research, development, and services, believing that progress is the only way to bring better experiences for clients.



SuperFile® Headquarters



WE
SPECIALIZE
IN ENDO.

Global Sales Network



The following diagram only represents some regions of our company's business sales network.



Super-Pro

SuperFile Patented System



- Gradient flexibility is produced with patented technology, presenting different flexibility.
- Progressive flexibility transition from the upper to the apical.
- Depth marking area helps the file to be well centered in the root canal and effectively conduct axial force source.
- Respect natural root canal anatomy, get accurate feedback from file tip with true expression of root canal morphology.
- Sequential shaping reduce the difficulty of preparing small and curved root canals
- One last standardized finishing file Z3 25/.06 for intend to taper confirmation and precise match to GP.

Product (ISO) Mode

SX
019/ .04v

T
017/ .02v

Z1
020/ .04v

Z2
020/ .06v

Z3
025/ .06v

* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 29.


Clinical Cases



* This dental tablet comes from customer feedback after using the product.

Super-Wave Gold

One File System

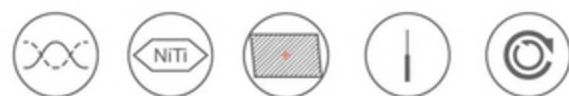

- MaxTech Niti alloy.
- Reciprocating movement.
- Powerful cuts for more efficient treatment.
- Progressive on taper design and cutting flute length.
- Crown-down technique, suitable for most root canals.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 29.

Clinical Cases



* This dental tablet comes from customer feedback after using the product.

Super-Wave Tunnel

- MaxTech 3.0 Niti alloy.
- Largely enhanced cyclic fatigue life.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 29.



RC-Series

One File System



RC-Series is of classic one-file system with extra flexibility, MaxTech6.0 Niti Alloy heat treatment characterized files with blue colorindicating its flexible feature to cover most canals morphology.

- MaxTech 1.0 3.0 6.0 Niti alloy.
- Suitable for normal/severe curved canal.
- More flexible and can be pre-shaped.
- Reciprocating motion.
- Regressive taper design to prevent unnecessary loss of coronal dentin.

• USAGE STEPS:

Establish the straightline access to canal orifice.

Choose the proper size V Blue according to X-ray film.

Shaping the canal by reciprocating 2/3. Better irrigate it every 3-4 pecks.


Repeating shaping until it reaches working length.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 29.

Clinical Cases

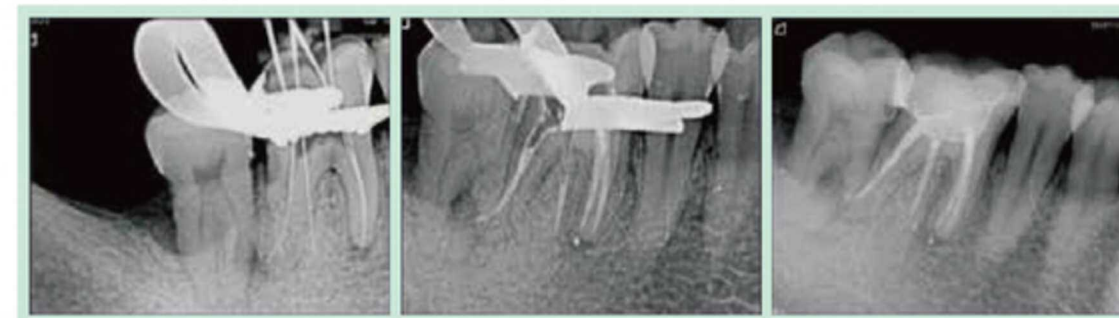

* This dental tablet comes from customer feedback after using the product.

Taper Next

Minimum Invasive System



Taper Next Series features with special Geometry design creating a swaggering trajectory to enlarge debris removal efficiency. Taper Next Series performance has been refined by Maxtech heated treatment and becomes more flexible to confront difficult narrow cases.

- MaxTech 6.0 NiTi alloy.
- Variable taper design to have accurate shaping.
- More flexibility with strong cutting ability.
- Minimum invasive simple sequence.

• USAGE STEPS:

Prepare straightline access to canal orifice.

Explore the canal with small size hand files (e.g. SS K-file 10).

Improve orifice access with openers (e.g. SX).

Glide the path with path files (e.g. T+File).

Shape the canal with X1 file by brushing in 3-4 passes until working length is reached.

Continue shaping with X2 until it reaches working length. If necessary, come to X3, X4, X5 for thick canals.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory. Purchase order code can be found on page 29.


Clinical Cases



* This dental tablet comes from customer feedback after using the product.

Master

Fixed Taper System

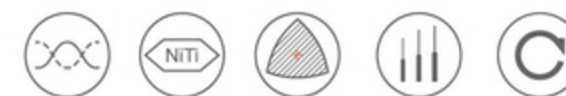

Master File is a very safe rotary file, and the fixed taper allows it to prepare effectively with minimal removal of dentin.


- MaxTech 6.0 NiTi alloy.
- Triangular cross section, with fixed taper .02/ .04/ .06 .
- Continuous rotation, 300~350 rpm.
- Shank: 13mm with taper marking rings.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory. Purchase order code can be found on page 29.

Clinical Cases

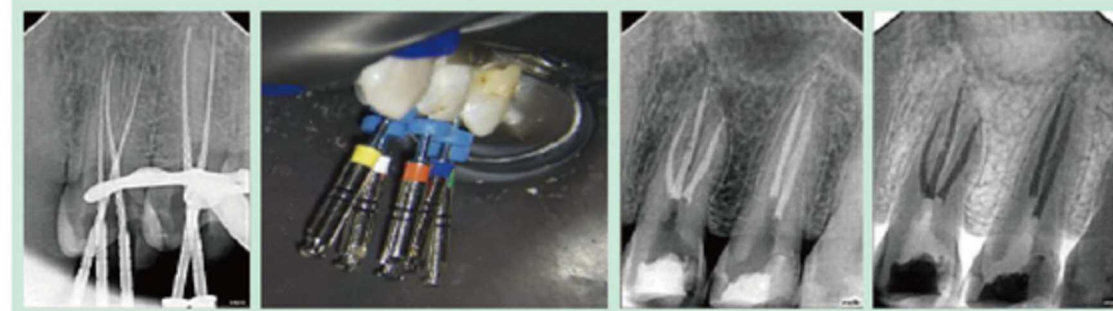

* This dental tablet comes from customer feedback after using the product.

RS File

Classic Rotation System



• Easy

- A. Easy to transfer from manual to rotary file system.
- B. One Sequence for all canals.
- C. Each files are reaching to working length.

• Safe

- A. 10/.04, 15/.05 worked as glide path, keep safe for following shaping.
- B. Small pitch design of small files confirm more cutting edges and increase instruments stability.
- C. Large files with progressive pitch design assure no screw-in effect.

• Efficient

- A. Special cross section design enable to improve cutting efficiency.
- B. Full range of size to cover more morphology.


Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 30.

Pedo File

Pedo System

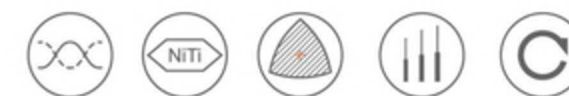

Design for root canal of deciduous teeth.

- Extremely soft to achieve a perfect preparation.
- Match the root canals original morphology.
- Fixed taper .04/ .06, brings accurate cutting and easier obturation.


Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 30.

Super-Pass

New Learner System

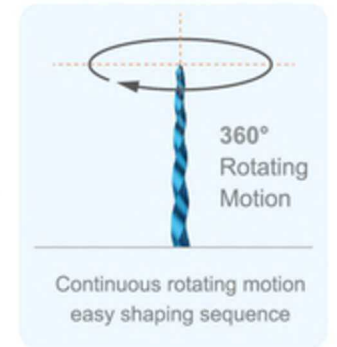

- MaxTech 6.0 controlled memory Nitinol alloy.
- Continuous rotating motion, easy shaping sequence.
- Convex triangle cross section brings great centering ability.
- Each file glide the path for the next one.
- Constant-tapered benefit simple obturation.

• USAGE STEPS:

Prepare straightline access to canal orifice.

Explore the canal with small size hand files (e.g. SS K-file 10).

Open 2/3 upper coronal part with opener P1 20/.07.

Shape and finish the canal with P2 15/.04, P3 20/.04, P4 20/.06, P5 25/.06 respectively to reach W/L passively by brushing.


EDTA, irrigation is recommended to be used for better result.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 30.

Clinical Cases

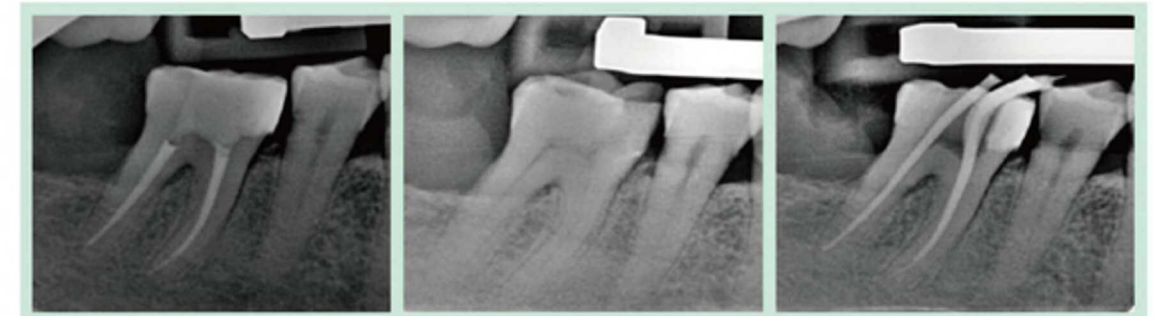

* This dental tablet comes from customer feedback after using the product.

S Taper

Classic Rotation System




- MaxTech Nitinol alloy (1.0, 3.0, 6.0).
- Suitable for regular canal preparation.
- Triangular cross section and safe guiding tip.
- Variable taper, optimize file function.
- Crown-Down Sequence:
Create straight-line access to canal orifice (SX).
Shape the coronal 2/3, confirm working length and smooth the root canal (S1, S2).
Shape the apical 1/3 with finisher (F1, F2). If necessary, proceed to F3, F4, F5
SX → S1 → S2 → F1/F2 → F3/F4/F5.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 30.

Clinical Cases

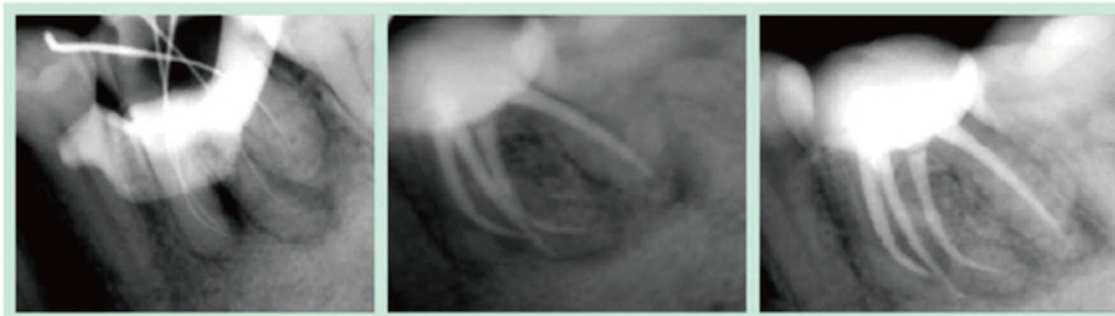

* This dental tablet comes from customer feedback after using the product.

Super-Large

New Learner System



- Super-Large is designed for large root canal preparation.
- Doctors can choose the appropriate based on the clinical root canal cases.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 30.

Super-One

One File System



- MaxTech 6.0 controlled memory NiTi alloy.
- Continuous rotating motion.
- Single file system (shaper and finisher).
- Regressive taper.
- Progressive screw pitches can hold more debris and lowerscrew-in risks.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 31.

Super-Can

Glide Path System



- Made with MaxTech 6.0 NiTi alloy.
- Glide path management technology, can swiftly create a smooth root canal path.
- Handle laser mark indicates specifications and models.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 31

TN File



- MaxTech 3.0 heat treated Niti files.
- Special instruments geometry for better debris removal.
- Respecting root canal anatomy.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 31.

S7 File

New Products



- Constant taper for easy obturation.
- Extremely flexible for following canal anatomy.
- Parabolic cross section.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 31.

CF File

New Products



- Convex Triangular cross section exhibits excellent centricity.
- Extremely flexible for following canal anatomy.
- Great resistance to cyclic fatigue.


Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 31.


S Taper Ultimate File

New Products

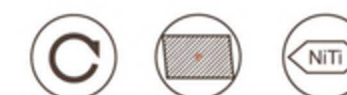

- Applicable to the full range of canal anatomy.
- Less screw-in effect and great cutting effect.
- Safer cross-section design.

Product (ISO) Mode

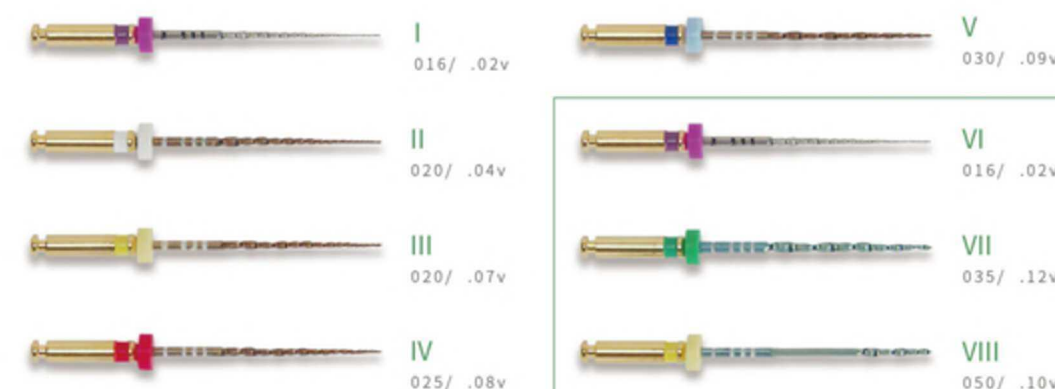

Large Canal

* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 32.

RT Series



- MaxTech Niti alloy.
- Progressive on taper design and cutting flute length.
- Remover of gutta percha and other obturation material.
- Color-coded rings correspond to different sizes.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 32.

Path File



- MaxTech NiTi alloy.
- Variable progressive taper.
- Triangular cross section.
- Respect the root canal anatomy.
- Create glide path and confirm root canal patency.
- Continuous clockwise rotation.
- Speed : 250 rpm Torque 2.0 N.cm.

Product (ISO) Mode



Path File
017



Path+ File
017

* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 31.

AS Series



- MaxTech NiTi alloy.
- Variable cross section, triangular shape at coronal, S shape at apical blade.
- One file system with continuous rotation.
- Accurate cutting.

Product (ISO) Mode




AS File
025



AS+ File
025

* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 32.

S Path

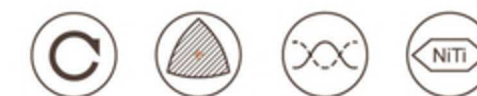

- MaxTech NiTi alloy.
- Variable progressive taper.
- Respect the root canal anatomy.
- Create glide path and confirm root canal patency.
- Continuous clockwise rotation.
- Speed : 250rpm Torque 3.0N.cm.

Product (ISO) Mode



S Path
015

* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 32.

Superfile



- MaxTech NiTi alloy.
- Convex triangular cross section.
- Progressive on taper design and cutting flute length.
- Silicon handle, color correspond to different file sizes.
- **Crown-Down Sequence:**
Create straight-line access to canal orifice (SX).
Shape the coronal 2/3, confirm working length and smooth the root canal (S1, S2).
Shape the apical 1/3 with finisher (F1, F2). If necessary, proceed to F3,
SX→S1→S2→F1/F2→F3.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on salesperson.

C File

SuperFile® stainless steel C-File is used for penetration of calcified canals.



Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 32.

K File



- Square cross section.
- Available in Niti and Stainless Steel material.
- Autoclavable.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 32.

H File



- Available in Niti Alloy and Stainless Steel material.
- Round cross section.
- Autoclavable.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 33.

K-Flex



Product (ISO) Mode




* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 33.

Reamer



Product (ISO) Mode

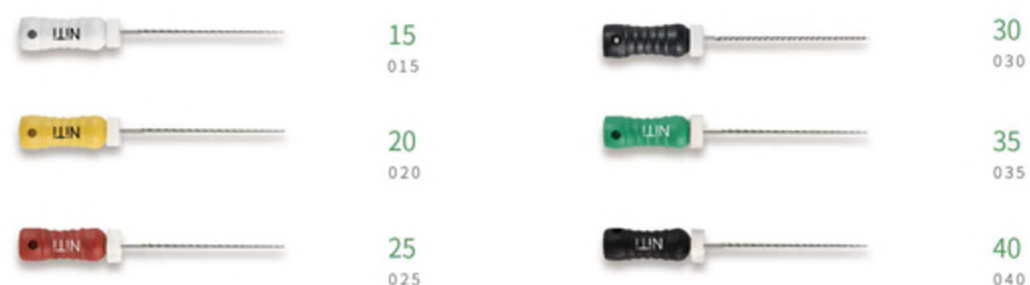

* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on salesperson.

Niti Engine Condenser



Product (ISO) Mode



Size

25/30/35/40

* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 33.

S.S. Rotary Paste Carrier



Product (ISO) Mode



Size

25/30/35/40

* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 32.

Spreader



- Used for filling gutta percha points in medium or thin curve canals laterally.
- With stopper to mark the working length.
- Autoclavable.

Product (ISO) Mode



Plugger



- Used for filling gutta percha points vertically in apical area or curve canals.
- With stopper to mark the working length.
- Autoclavable.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Spreader&Plugger Purchase order code can be found on page 33.

Smooth Broach

Smooth Broach is used for absorbing moisture in the root canal and for carrying disinfectant. It is easy to wind a wad of absorbent cotton around it.

Product (ISO) Mode
Size
0/2/4

* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 34.



Square Broach

Square Broach is a hand-used instrument for removing pulp tissue during root canal treatments.

Product (ISO) Mode
Size
000/00/0


* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 34.



Barbed Broach

Barbed Broach is a hand-used instrument for removing pulp tissue during root canal treatments.

- Instruments with barbs for removing soft tissue from root canals.
- Used for taking out dental pulp by turning 180 degrees after entering the root canals.

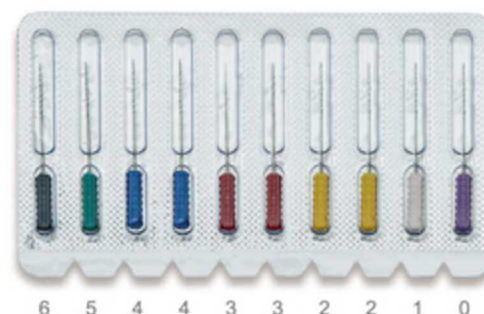

* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 33.

Endo Debrider

H-Files type instruments are used for debriding canals under microscope.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 34.

Endo Opener

K-Files type instruments are used for opening canals under microscope.

Product (ISO) Mode



* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 34.

Gates Drill



Gates drill used for expanding the canal entrance or preparation for middle-upper part of the canals.

- Medical Stainless Steel.
- Corresponding sizing rings.
- Sizes: 50 70 90 110 130 150.
- Recommended speed: 800-1200 rpm.

Product (ISO) Mode



Size: 50
Rings: 1



Size: 110
Rings: 4



Size: 70
Rings: 2



Size: 130
Rings: 5



Size: 90
Rings: 3



Size: 150
Rings: 6

* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 34.

Peeso Reamer



Peeso Reamer used for preparing the 1/3 coronal part of canals, obturation removal and post preparation.

- Medical Stainless Steel.
- Corresponding sizing rings.
- Working length of blades: 0.70mm - 1.70mm.
- Sizes: 70 90 110 130 150 170.

Product (ISO) Mode



Size: 50
Rings: 1



Size: 110
Rings: 4



Size: 70
Rings: 2



Size: 130
Rings: 5



Size: 90
Rings: 3



Size: 150
Rings: 6

* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 34.

GP Plugger



GP Plugger is used for warm and cold condensation.

- Available in Niti and stainless steel.
- Color-coded handle.
- Light weight.

Product (ISO) Mode



Size: 35-70



Size: 50-100



Size: 40-80



Size: 60-120

* All the above data is sourced from **SuperFile®** Laboratory.
Purchase order code can be found on page 34.

Purchase order code

Super-pro				
	19mm	21mm	25mm	31mm
SX	GFF 019 1904	-	-	-
T	-	GFF 021 1702	GFF 025 1702	GFF 031 1702
Z1	-	GFF 021 2004	GFF 025 2004	GFF 031 2004
Z2	-	GFF 021 2006	GFF 025 2006	GFF 031 2006
Z3	-	GFF 021 2506	GFF 025 2506	GFF 031 2506
SX-Z3	-	GFF 021 A001	GFF 025 A001	GFF 031 A001

Super-Wave Gold				
Size	21mm	25mm	31mm	
20	AFW 021 020	AFW 025 020	AFW 031 020	
25	AFW 021 025	AFW 025 025	AFW 031 025	
35	AFW 021 035	AFW 025 035	AFW 031 035	
45	AFW 021 045	AFW 025 045	AFW 031 045	
20/25/35/45	AFW 021 A01	AFW 025 A01	AFW 031 A01	

Super-Wave Tunnel				
Size	21mm	25mm	31mm	
15	AFW 021 015	AFW 025 015	AFW 031 015	

Master 2				
Size	21mm	25mm	31mm	
15	BK2 021 015	BK2 025 015	BK2 031 015	
20	BK2 021 020	BK2 025 020	BK2 031 020	
25	BK2 021 025	BK2 025 025	BK2 031 025	
30	BK2 021 030	BK2 025 030	BK2 031 030	
35	BK2 021 035	BK2 025 035	BK2 031 035	
40	BK2 021 040	BK2 025 040	BK2 031 040	
15-40	BK2 021 A01	BK2 025 A01	BK2 031 A01	

Master 4				
Size	21mm	25mm	31mm	
15	BK4 021 015	BK4 025 015	BK4 031 015	
20	BK4 021 020	BK4 025 020	BK4 031 020	
25	BK4 021 025	BK4 025 025	BK4 031 025	
30	BK4 021 030	BK4 025 030	BK4 031 030	
35	BK4 021 035	BK4 025 035	BK4 031 035	
40	BK4 021 040	BK4 025 040	BK4 031 040	
15-40	BK4 021 A01	BK4 025 A01	BK4 031 A01	

Master 6				
Size	21mm	25mm	31mm	
15	BK6 021 015	BK6 025 015	BK6 031 015	
20	BK6 021 020	BK6 025 020	BK6 031 020	
25	BK6 021 025	BK6 025 025	BK6 031 025	
30	BK6 021 030	BK6 025 030	BK6 031 030	
35	BK6 021 035	BK6 025 035	BK6 031 035	
40	BK6 021 040	BK6 025 040	BK6 031 040	
15-40	BK6 021 A01	BK6 025 A01	BK6 031 A01	

RC-Series				
Size	21mm	25mm	31mm	
25	SFV 021 025	SFV 025 025	SFV 031 025	
40	SFV 021 040	SFV 025 040	SFV 031 040	
50	SFV 021 050	SFV 025 050	SFV 031 050	
25/40/50	SFV 021 A01	SFV 025 A01	SFV 031 A01	

RC-Series				
Size	21mm	25mm	31mm	
25	AFV 021 025	AFV 025 025	AFV 031 025	
40	AFV 021 040	AFV 025 040	AFV 031 040	
50	AFV 021 050	AFV 025 050	AFV 031 050	
25/40/50	AFV 021 A01	AFV 025 A01	AFV 031 A01	

RC-Series				
Size	21mm	25mm	31mm	
25	BFV 021 025	BFV 025 025	BFV 031 025	
40	BFV 021 040	BFV 025 040	BFV 031 040	
50	BFV 021 050	BFV 025 050	BFV 031 050	
25/40/50	BFV 021 A01	BFV 025 A01	BFV 031 A01	

Taper Next Series				
Model	21mm	25mm	31mm	
X1	AFX 021 0X1	AFX 025 0X1	AFX 031 0X1	
X2	AFX 021 0X2	AFX 025 0X2	AFX 031 0X2	
X3	AFX 021 0X3	AFX 025 0X3	AFX 031 0X3	
X4	AFX 021 0X4	AFX 025 0X4	AFX 031 0X4	
X5	AFX 021 0X5	AFX 025 0X5	AFX 031 0X5	
X1-X3	AFX 021 A01	AFX 025 A01	AFX 031 A01	

Taper Next Series				
Model	21mm	25mm	31mm	
X1	BFX 021 0X1	BFX 025 0X1	BFX 031 0X1	
X2	BFX 021 0X2	BFX 025 0X2	BFX 031 0X2	
X3	BFX 021 0X3	BFX 025 0X3	BFX 031 0X3	
X4	BFX 021 0X4	BFX 025 0X4	BFX 031 0X4	
X5	BFX 021 0X5	BFX 025 0X5	BFX 031 0X5	
X1-X3	BFX 021 A01	BFX 025 A01	BFX 031 A01	

Taper Next Series				
Model	21mm	25mm	31mm	
X1	CFX 021 0X1	CFX 025 0X1	CFX 031 0X1	
X2	CFX 021 0X2	CFX 025 0X2	CFX 031 0X2	
X3	CFX 021 0X3	CFX 025 0X3	CFX 031 0X3	
X4	CFX 021 0X4	CFX 025 0X4	CFX 031 0X4	
X5	CFX 021 0X5	CFX 025 0X5	CFX 031 0X5	
X1-X3	CFX 021 A01	CFX 025 A01	CFX 031 A01	

RS File				
	21mm	25mm	31mm	
10/04	BKM 021 104	BKM 025 104	BKM 031 104	
15/05	BKM 021 155	BKM 025 155	BKM 031 155	
20/06	BKM 021 206	BKM 025 206	BKM 031 206	
25/06	BKM 021 256	BKM 025 256	BKM 031 256	
25/07	BKM 021 257	BKM 025 257	BKM 031 257	
30/05	BKM 021 305	BKM 025 305	BKM 031 305	
35/04	BKM 021 354	BKM 025 354	BKM 031 354	
40/04	BKM 021 404	BKM 025 404	BKM 031 404	
10/04-25/07	BKM 021 A01	BKM 025 A01	BKM 031 A01	

Pedo 4 File		
20	ADT 004 020	
25	ADT 004 025	
30	ADT 004 030	
35	ADT 004 035	
20-35	ADT 004 A01	

Pedo 4 File		
20	BDT 004 020	
25	BDT 004 025	
30	BDT 004 030	
35	BDT 004 035	
20-35	BDT 004 A01	

S Taper Series				
	19mm	21mm	25mm	31mm
SX	AFU 019 0SX	-	-	-
S1	-	AFU 021 0S1	AFU 025 0S1	AFU 031 0S1
S2	-	AFU 021 0S2	AFU 025 0S2	AFU 031 0S2
F1	-	AFU 021 0F1	AFU 025 0F1	AFU 031 0F1
F2	-	AFU 021 0F2	AFU 025 0F2	AFU 031 0F2
F3	-	AFU 021 0F3	AFU 025 0F3	AFU 031 0F3
F4	-	AFU 021 0F4	AFU 025 0F4	AFU 031 0F4
F5	-	AFU 021 0F5	AFU 025 0F5	AFU 031 0F5
SX-F3	-	AFU 021 A01	AFU 025 A01	AFU 031 A01

S Taper Series				
	19mm	21mm	25mm	31mm
SX	BFU 019 0SX	-	-	-
S1	-	BFU 021 0S1	BFU 025 0S1	BFU 031 0S1
S2	-	BFU 021 0S2	BFU 025 0S2	BFU 031 0S2
F1	-	BFU 021 0F1	BFU 025 0F1	BFU 031 0F1
F2	-	BFU 021 0F2	BFU 025 0F2	BFU 031 0F2
F3	-	BFU 021 0F3	BFU 025 0F3	BFU 031 0F3
F4	-	BFU 021 0F4	BFU 025 0F4	BFU 031 0F4
F5	-	BFU 021 0F5	BFU 025 0F5	BFU 031 0F5
SX-F3	-	BFU 021 A01	BFU 025 A01	BFU 031 A01

Pedo 6 File		
20	ADT 006 020	
25	ADT 006 025	
30	ADT 006 030	
35	ADT 006 035	
20-35	ADT 006 A01	

Pedo 6 File		
20	BDT 006 020	
25	BDT 006 025	
30	BDT 006 030	
35	BDT 006 035	
20-35	BDT 006 A01	

Super-Pass				
	19mm	21mm	25mm	31mm
20/07	EZP 019 2007	-	-	-
15/04	-	EZP 021 1504	EZP 025 1504	EZP 031 1504
20/04	-	EZP 021 2004	EZP 025 2004	EZP 031 2004
20/06	-	EZP 021 2006	EZP 025 2006	EZP 031 2006
25/04	-	EZP 021 2504	EZP 025 2504	EZP 031 2504
25/06	-	EZP 021 2506	EZP 025 2506	EZP 031 2506
Sorted	-	EZP 021 A001	EZP 025 A001	EZP 031 A001

Super-Large				
	21mm	25mm	31mm	
L1	EZL 021 3004	EZL 025 3004	EZL 031 3004	
L2	EZL 021 3006	EZL 025 3006	EZL 031 3006	
L3	EZL 021 3504	EZL 025 3504	EZL 031 3504	
L4	EZL 021 3506	EZL 025 3506	EZL 031 3506	
L5	EZL 021 4004	EZL 025 4004	EZL 031 4004	
L6	EZL 021 5004	EZL 025 5004	EZL 031 5004	

Purchase order code

Super-One

Size	21mm	25mm	31mm
● Small	EZO 021 2007	EZO 025 2007	EZO 031 2007
● Primary	EZO 021 2507	EZO 025 2507	EZO 031 2507
● Medium	EZO 021 3506	EZO 025 3506	EZO 031 3506
○ Large	EZO 021 4505	EZO 025 4505	EZO 031 4505

Super-Can

	17mm	21mm	25mm	31mm
● Opener	EZC 017 2007	-	-	-
● Glide Path I	-	EZC 021 1004	EZC 025 1004	EZC 031 1004
○ Glide Path II	-	EZC 021 1502	EZC 025 1502	EZC 031 1502

TN File

	16mm	21mm	25mm	31mm
20/.08	ATT 016 020	-	-	-
17/.02	-	ATT 021 017	ATT 025 017	ATT 031 017
20/.04	-	ATT 021 020	ATT 025 020	ATT 031 020
26/.04	-	ATT 021 026	ATT 025 026	ATT 031 026
36/.04	-	ATT 021 036	ATT 025 036	ATT 031 036
20/.08-36/.04	-	ATT 021 A01	ATT 025 A01	ATT 031 A01

S7 File

	21mm	25mm	31mm
17/.04	AMH4 021 017	AMH4 025 017	AMH4 031 017
20/.04	AMH4 021 020	AMH4 025 020	AMH4 031 020
25/.04	AMH4 021 025	AMH4 025 025	AMH4 031 025
30/.04	AMH4 021 030	AMH4 025 030	AMH4 031 030
35/.04	AMH4 021 035	AMH4 025 035	AMH4 031 035
40/.04	AMH4 021 040	AMH4 025 040	AMH4 031 040
17/.04-40/.04	AMH4 021 A01	AMH4 025 A01	AMH4 031 A01

CF 4 File

Size	21mm	25mm	31mm
○ 15	BKD4 021 015	BKD4 025 015	BKD4 031 015
● 20	BKD4 021 020	BKD4 025 020	BKD4 031 020
● 25	BKD4 021 025	BKD4 025 025	BKD4 031 025
● 30	BKD4 021 030	BKD4 025 030	BKD4 031 030
● 35	BKD4 021 035	BKD4 025 035	BKD4 031 035
● 40	BKD4 021 040	BKD4 025 040	BKD4 031 040
15-40	BKD4 021 A01	BKD4 025 A01	BKD4 031 A01

Path File

Size	21mm	25mm	31mm
17	SFT 021 017	SFT 025 017	SFT 031 017

S7 File

	21mm	25mm	31mm
17/.06	AMH6 021 017	AMH6 025 017	AMH6 031 017
20/.06	AMH6 021 020	AMH6 025 020	AMH6 031 020
25/.06	AMH6 021 025	AMH6 025 025	AMH6 031 025
30/.06	AMH6 021 030	AMH6 025 030	AMH6 031 030
35/.06	AMH6 021 035	AMH6 025 035	AMH6 031 035
40/.06	AMH6 021 040	AMH6 025 040	AMH6 031 040
17/.06-40/.06	AMH6 021 A01	AMH6 025 A01	AMH6 031 A01

CF 6 File

Size	21mm	25mm	31mm
○ 15	BKD6 021 015	BKD6 025 015	BKD6 031 015
● 20	BKD6 021 020	BKD6 025 020	BKD6 031 020
● 25	BKD6 021 025	BKD6 025 025	BKD6 031 025
● 30	BKD6 021 030	BKD6 025 030	BKD6 031 030
● 35	BKD6 021 035	BKD6 025 035	BKD6 031 035
● 40	BKD6 021 040	BKD6 025 040	BKD6 031 040
15-40	BKD6 021 A01	BKD6 025 A01	BKD6 031 A01

Path File

Size	21mm	25mm	31mm
17	AFT 021 017	AFT 025 017	AFT 031 017

S Taper Ultimate File

	21mm	25mm	31mm
16/.02	AFZ 021 1602	AFZ 025 1602	AFZ 031 1602
20/.04	AFZ 021 2004	AFZ 025 2004	AFZ 031 2004
20/.07	AFZ 021 2007	AFZ 025 2007	AFZ 031 2007
25/.08	AFZ 021 2508	AFZ 025 2508	AFZ 031 2508
30/.09	AFZ 021 3009	AFZ 025 3009	AFZ 031 3009
35/.12	AFZ 021 3512	AFZ 025 3512	AFZ 031 3512
50/.10	AFZ 021 5010	AFZ 025 5010	AFZ 031 5010
16/.02-35/.12	AFZ 021 A01	AFZ 025 A01	AFZ 031 A01

AS Series

Size	21mm	25mm	29mm
25	SFA 021 025	SFA 025 025	SFA 029 025

AS Series

Size	21mm	25mm	29mm
25	AFA 021 025	AFA 025 025	AFA 029 025

Rotary Paste Carrier

Size	21mm	25mm	29mm
● 25	EPC 021 025	EPC 025 025	EPC 029 025
● 30	EPC 021 030	EPC 025 030	EPC 029 030
● 35	EPC 021 035	EPC 025 035	EPC 029 035
● 40	EPC 021 040	EPC 025 040	EPC 029 040
25-40	EPC 021 A01	EPC 025 A01	EPC 029 A01

S.S. Hand Use K-File

	21mm	25mm	28mm	31mm	21mm	25mm	28mm	31mm
● 6	HSK 021 006	HSK 025 006	HSK 028 006	HSK 031 006	-	-	-	-
● 8	HSK 021 008	HSK 025 008	HSK 028 008	HSK 031 008	HNK 021 008	HNK 025 008	HNK 028 008	HNK 031 008
● 10	HSK 021 010	HSK 025 010	HSK 028 010	HSK 031 010	HNK 021 010	HNK 025 010	HNK 028 010	HNK 031 010
○ 15	HSK 021 015	HSK 025 015	HSK 028 015	HSK 031 015	HNK 021 015	HNK 025 015	HNK 028 015	HNK 031 015
● 20	HSK 021 020	HSK 025020	HSK 028 020	HSK 031 020	HNK 021 020	HNK 025020	HNK 028 020	HNK 031 020
● 25	HSK 021 025	HSK 025 025	HSK 025 025	HSK 031 025	HNK 021 025	HNK 025 025	HNK 025 025	HNK 031 025
● 30	HSK 021 030	HSK 025 030	HSK 025 030	HSK 031 030	HNK 021 030	HNK 025 030	HNK 025 030	HNK 031 030
● 35	HSK 021 035	HSK 025 035	HSK 025 035	HSK 031 035	HNK 021 035	HNK 025 035	HNK 025 035	HNK 031 035
● 40	HSK 021 040	HSK 025 040	HSK 025 040	HSK 031 040	HNK 021 040	HNK 025 040	HNK 025 040	HNK 031 040
○ 45	HSK 021 045	HSK 025 045	HSK 025 045	HSK 031 045	HNK 021 045	HNK 025 045	HNK 025 045	HNK 031 045
● 50	HSK 021 050	HSK 025 050	HSK 025 050	HSK 031 050	HNK 021 050	HNK 025 050	HNK 025 050	HNK 031 050
● 55	HSK 021 055	HSK 025 055	HSK 025 055	HSK 031 055	HNK 021 055	HNK 025 055	HNK 025 055	HNK 031 055
● 60	HSK 021 060	HSK 025 060	HSK 025 060	HSK 031 060	HNK 021 060	HNK 025 060	HNK 025 060	HNK 031 060
● 70	HSK 021 070	HSK 025 070	HSK 025 070	HSK 031 070	HNK 021 070	HNK 025 070	HNK 025 070	HNK 031 070
● 80	HSK 021 080	HSK 025 080	HSK 025 080	HSK 031 080	HNK 021 080	HNK 025 080	HNK 025 080	HNK 031 080
15-40	HSK 021 A01	HSK 025 A01	HSK 025 A01	HSK 031 A01	HNK 021 A01	HNK 025 A01	HNK 025 A01	HNK 031 A01
45-80	HSK 021 A02	HSK 025 A02	HSK 025 A02	HSK 031 A02	HNK 021 A02	HNK 025 A02	HNK 025 A02	HNK 031 A02

RT Series

	16mm	18mm	22mm
D1	SFO 016 0D1	-	-
D2	-	SFO 018 0D2	-
D3	-	-	SFO 022 0D2
D1-D3	SFO 000 A01	-	-

RT Series

	16mm	18mm	22mm
D1	AFO 016 0D1	-	-
D2	-	AFO 018 0D2	-
D3	-	-	AFO 022 0D3
D1-D3	AFO 000 A01	-	-

S Path

Size	21mm	25mm	31mm
16	SFP 021 016	SFP 025 016	SFP 031 016

S.S. C-File

	21mm	25mm	28mm
● 6	HSC 021 006	HSC 025 006	HSC 028 006
● 8	HSC 021 008	HSC 025 008	HSC 028 008
● 10	HSC 021 010	HSC 025 010	HSC 028 010
● 12	HSC 021 012	HSC 025 012	HSC 028 012
○ 15	HSC 021 015	HSC 025 015	HSC 028 015
8-15	HSC 021 A01	HSC 025 A01	HSC 028 A01
6-10	HSC 021 A02	HSC 025 A02	HSC 028 A02

NiTi Hand Use K-File

	21mm	25mm	28mm	31mm
● 6	HSK 021 006	HSK 025 006	HSK 028 006	HSK 031 006
● 8	HSK 021 008	HSK 025 008	HSK 028 008	HSK 031 008
● 10	HSK 021 010	HSK 025 010	HSK 028 010	HSK 031 010
○ 15	HSK 021 015	HSK 025 015	HSK 028 015	HSK 031 015
● 20	HSK 021 020	HSK 025020	HSK 028 020	HSK 031 020
● 25	HSK 021 025	HSK 025 025	HSK 025 025	HSK 031 025
● 30	HSK 021 030	HSK 025 030	HSK 025 030	HSK 031 030
● 35	HSK 021 035	HSK 025 035	HSK 025 035	HSK 031 035
● 40	HSK 021 040	HSK 025 040	HSK 025 040	HSK 031 040
○ 45	HSK 021 045	HSK 025 045	HSK 025 045	HSK 031 045
● 50	HSK 021 050	HSK 025 050	HSK 025 050	HSK 031 050
● 55	HSK 021 055	HSK 025 055	HSK 025 055	HSK 031 055
● 60	HSK 021 060	HSK 025 060	HSK 025 060	HSK 031 060
● 70	HSK 021 070	HSK 025 070	HSK 025 070	HSK 031 070
● 80	HSK 021 080	HSK 025 080	HSK 025 080	HSK 031 080
15-40	HSK 021 A01	HSK 025 A01	HSK 025 A01	HSK 031 A01
45-80	HSK 021 A02	HSK 025 A02	HSK 025 A02	HSK 031 A02

Purchase order code

S.S. Hand Use H-File

	21mm	25mm	28mm	31mm
6	SHS 021 006	SHS 025 006	SHS 028 006	SHS 031 006
8	SHS 021 008	SHS 025 008	SHS 028 008	SHS 031 008
10	SHS 021 010	SHS 025 010	SHS 028 010	SHS 031 010
15	SHS 021 015	SHS 025 015	SHS 028 015	SHS 031 015
20	SHS 021 020	SHS 025 020	SHS 028 020	SHS 031 020
25	SHS 021 025	SHS 025 025	SHS 025 025	SHS 031 025
30	SHS 021 030	SHS 025 030	SHS 025 030	SHS 031 030
35	SHS 021 035	SHS 025 035	SHS 025 035	SHS 031 035
40	SHS 021 040	SHS 025 040	SHS 025 040	SHS 031 040
45	SHS 021 045	SHS 025 045	SHS 025 045	SHS 031 045
50	SHS 021 050	SHS 025 050	SHS 025 050	SHS 031 050
55	SHS 021 055	SHS 025 055	SHS 025 055	SHS 031 055
60	SHS 021 060	SHS 025 060	SHS 025 060	SHS 031 060
70	SHS 021 070	SHS 025 070	SHS 025 070	SHS 031 070
80	SHS 021 080	SHS 025 080	SHS 025 080	SHS 031 080
15~40	SHS 021 A01	SHS 025 A01	SHS 025 A01	SHS 031 A01
45~80	SHS 021 A02	SHS 025 A02	SHS 025 A02	SHS 031 A02

NiTi Hand Use H-File

	21mm	25mm	28mm	31mm
-	HNH 021 008	HNH 025 008	HNH 028 008	HNH 031 008
-	HNH 021 010	HNH 025 010	HNH 028 010	HNH 031 010
-	HNH 021 015	HNH 025 015	HNH 028 015	HNH 031 015
-	HNH 021 020	HNH 025 020	HNH 028 020	HNH 031 020
-	HNH 021 025	HNH 025 025	HNH 025 025	HNH 031 025
-	HNH 021 030	HNH 025 030	HNH 025 030	HNH 031 030
-	HNH 021 035	HNH 025 035	HNH 025 035	HNH 031 035
-	HNH 021 040	HNH 025 040	HNH 025 040	HNH 031 040
-	HNH 021 045	HNH 025 045	HNH 025 045	HNH 031 045
-	HNH 021 050	HNH 025 050	HNH 025 050	HNH 031 050
-	HNH 021 055	HNH 025 055	HNH 025 055	HNH 031 055
-	HNH 021 060	HNH 025 060	HNH 025 060	HNH 031 060
-	HNH 021 070	HNH 025 070	HNH 025 070	HNH 031 070
-	HNH 021 080	HNH 025 080	HNH 025 080	HNH 031 080
-	HNH 021 A01	HNH 025 A01	HNH 025 A01	HNH 031 A01
-	HNH 021 A02	HNH 025 A02	HNH 025 A02	HNH 031 A02

K-Flex

	21mm	25mm	28mm	31mm
15	TFK 021 015	TFK 025 015	TFK 028 015	TFK 031 015
20	TFK 021 020	TFK 025 020	TFK 028 020	TFK 031 020
25	TFK 021 025	TFK 025 025	TFK 028 025	TFK 031 025
30	TFK 021 030	TFK 025 030	TFK 028 030	TFK 031 030
35	TFK 021 035	TFK 025 035	TFK 028 035	TFK 031 035
40	TFK 021 040	TFK 025 040	TFK 028 040	TFK 031 040
15~40	TFK 021 A01	TFK 025 A01	TFK 028 A01	TFK 031 A01

Spreader

Size	21mm NiTi	25mm NiTi	21mm S.S.	25mm S.S.
15	HNS 021 015	HNS 025 015	HSS 021 015	HSS 025 015
20	HNS 021 020	HNS 025 020	HSS 021 020	HSS 025 020
25	HNS 021 025	HNS 025 025	HSS 021 025	HSS 025 025
30	HNS 021 030	HNS 025 030	HSS 021 030	HSS 025 030
35	HNS 021 035	HNS 025 035	HSS 021 035	HSS 025 035
40	HNS 021 040	HNS 025 040	HSS 021 040	HSS 025 040
15~40	HNS 021 A01	HNS 025 A01	HSS 021 A01	HSS 025 A01

Plugger

Size	21mm NiTi	25mm NiTi	21mm S.S.	25mm S.S.
15	HNP 021 015	HNP 025 015	HSP 021 015	HSP 025 015
20	HNP 021 020	HNP 025 020	HSP 021 020	HSP 025 020
25	HNP 021 025	HNP 025 025	HSP 021 025	HSP 025 025
30	HNP 021 030	HNP 025 030	HSP 021 030	HSP 025 030
35	HNP 021 035	HNP 025 035	HSP 021 035	HSP 025 035
40	HNP 021 040	HNP 025 040	HSP 021 040	HSP 025 040
15~40	HNP 021 A01	HNP 025 A01	HSP 021 A01	HSP 025 A01

.02 Niti Engine Condenser

Size	21mm	25mm
25	MC2 021 025	MC2 025 025
30	MC2 021 030	MC2 025 030
35	MC2 021 035	MC2 025 035
40	MC2 021 040	MC2 025 040
25~40	MC2 021 A01	MC2 025 A01

.04 Niti Engine Condenser

Size	21mm	25mm
25	MC4 021 025	MC4 025 025
30	MC4 021 030	MC4 025 030
35	MC4 021 035	MC4 025 035
40	MC4 021 040	MC4 025 040
25~40	MC4 021 A01	MC4 025 A01

Barbed Broach

Number	21mm	25mm
0	HSB 021 020	HSB 025 020
1	HSB 021 025	HSB 025 025
2	HSB 021 030	HSB 025 030
3	HSB 021 035	HSB 025 035
4	HSB 021 040	HSB 025 040
5	HSB 021 050	HSB 025 050
6	HSB 021 060	HSB 025 060
2~4	HSB 021 A01	HSB 025 A01
0~6	HSB 021 A02	HSB 025 A02

Smooth Broach

Size	52mm
0	HSB 052 010
2	HSB 052 030
4	HSB 052 050

Square Broach

Size	52mm
000	HSS 052 010
00	HSS 052 020
0	HSS 052 030

Endo Debrider

Size	-
20/02	HED T02 020
30/02	HED T02 030
SET(2pcs)	HED 000 A01

Endo Opener

Size	-
10/04	HED T04 010
15/04	HED T04 015
10/06	HED T06 010
SET(3pcs)	HED 000 B01

Gates Drill

Size	Rings	28mm	32mm
50	1	ESG 028 001	ESG 032 001
70	2	ESG 028 002	ESG 032 002
90	3	ESG 028 003	ESG 032 003
110	4	ESG 028 004	ESG 032 004
130	5	ESG 028 005	ESG 032 005
150	6	ESG 028 006	ESG 032 006
50-150	1-6	ESG 028 A01	ESG 032 A01

GP Plugger

Size	-
35-70	GPP 035 070
40-80	GPP 040 080
50-100	GPP 050 100
60-120	GPP 060 120
set 1 (3pcs)	GPP 000 A02
set 2 (4pcs)	GPP 000 A01

Peeso Reamer

Size	Rings	28mm	32mm
70	1	ESP 028 001	ESP 032 001
90	2	ESP 028 002	ESP 032 002
110	3	ESP 028 003	ESP 032 003
130	4	ESP 028 004	ESP 032 004
150	5	ESP 028 005	ESP 032 005
170	6	ESP 028 006	ESP 032 006
70-170	1-6	ESP 028 A01	ESP 032 A01